


## Neonatal Transilluminator Instructions

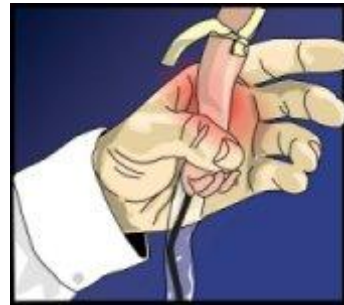

### Instructions for use.

- Apply A Tourniquet
- Dim the ambient lighting
- Place the light in the palm of your hand and position the infant's arm or foot over the light holding it in place with your thumb.
- Look on the top of the hand or foot to locate the dark lines which are vein.
- Verify the dark line is a vein by depressing it and watch it refill
- Proceed with the venipuncture as usual

### For Arteries

- Place the light on the top of the wrist area and look for the pulsing action on the other side of the wrist.
- Arteries do not absorb light so they will not present a dark line.



Graykon Scientific  
P.O. Box 200 Mont Albert  
Vic. 3127  
PH: 61 3 98989393  
Fax: 61 3 98989394  
Email: [sales@graykon.com.au](mailto:sales@graykon.com.au)  
Web: [www.graykon.com.au](http://www.graykon.com.au)

